


THE ONE AND  
ONLY WITH

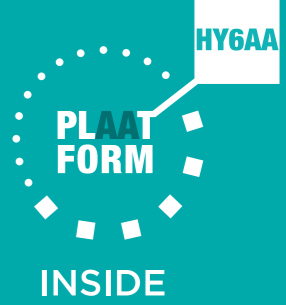


sune  
KOS


Body

To regenerate the  
ECM counteracting  
the body's skin laxity

Sunekos Body is an injectable indicated for the treatment of **striae distensae** and **skin laxity** of

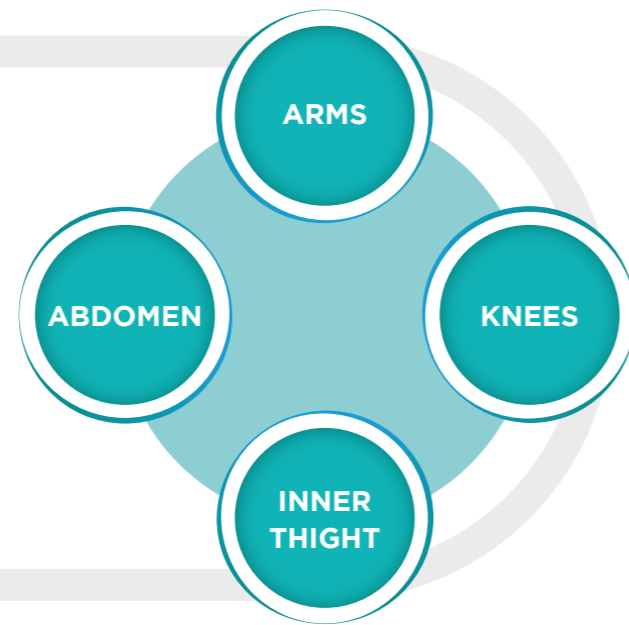

The dermis is different dependant on the region of the body.

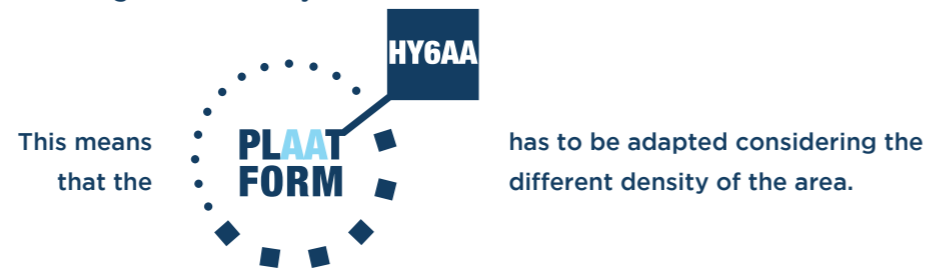


## Injection Technique




- Needle 27-29G and 12-13mm
- Medium-Deep Dermis injection
- 30°- 45° Retrograde linear injection
- Series of bolus at a distance of about 1.5cm
- 0.1 - 0.2 ml per bolus
- Injection in the direction of the flow (vs lymph nodes)
- Massage following the flow



INNER THIGH AND KNEES LAXITY

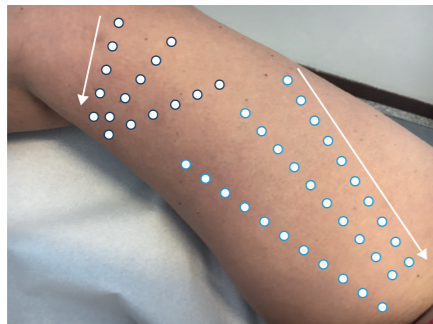

ARMS SEVERE LAXITY

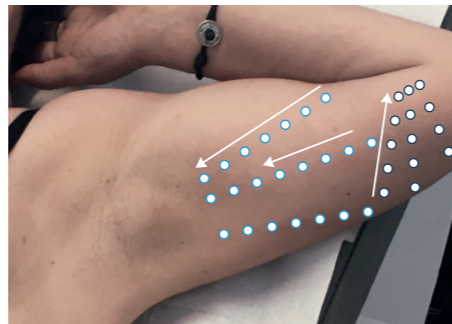

## Standard Protocol

- The amount of product is defined to treat:
- one of the specific areas in case of **severe situations**
  - two areas in case of **mild laxity** (i.e. inner arms and knees).




**If using in combination with laser treatment, please ensure there is a 7 day gap between the 2 different treatments.**

**Repeat every 6 months.**

## Clinical Evaluation

21 patients injected 4 times, once every week using only 5 ml on arms and 5 ml on legs. This means that the trial has been performed using only 1/2 vial per area.

### SKIN ROUGHNESS

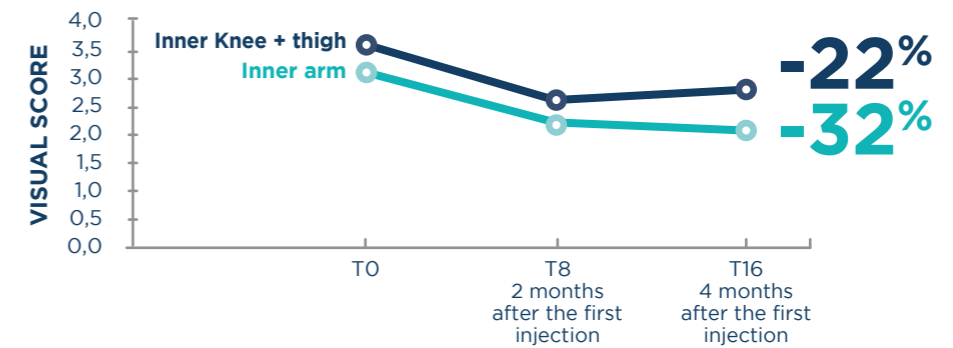

### SKIN LAXITY

Improvement corresponding to a reduction of the visual score of at least 1 grade




improvement measured on photographic scale

### SKIN DENSITY (profilometry)

| Area                        | T8      | T16     |
|-----------------------------|---------|---------|
| Rv (wrinkles maximum depth) | -24.1%* | -17.6%* |

\* Holm-Sidak Adjusted t test p<0.05 vs T0

Measured with Primos compact portable (GF Messtechnik) equipment.

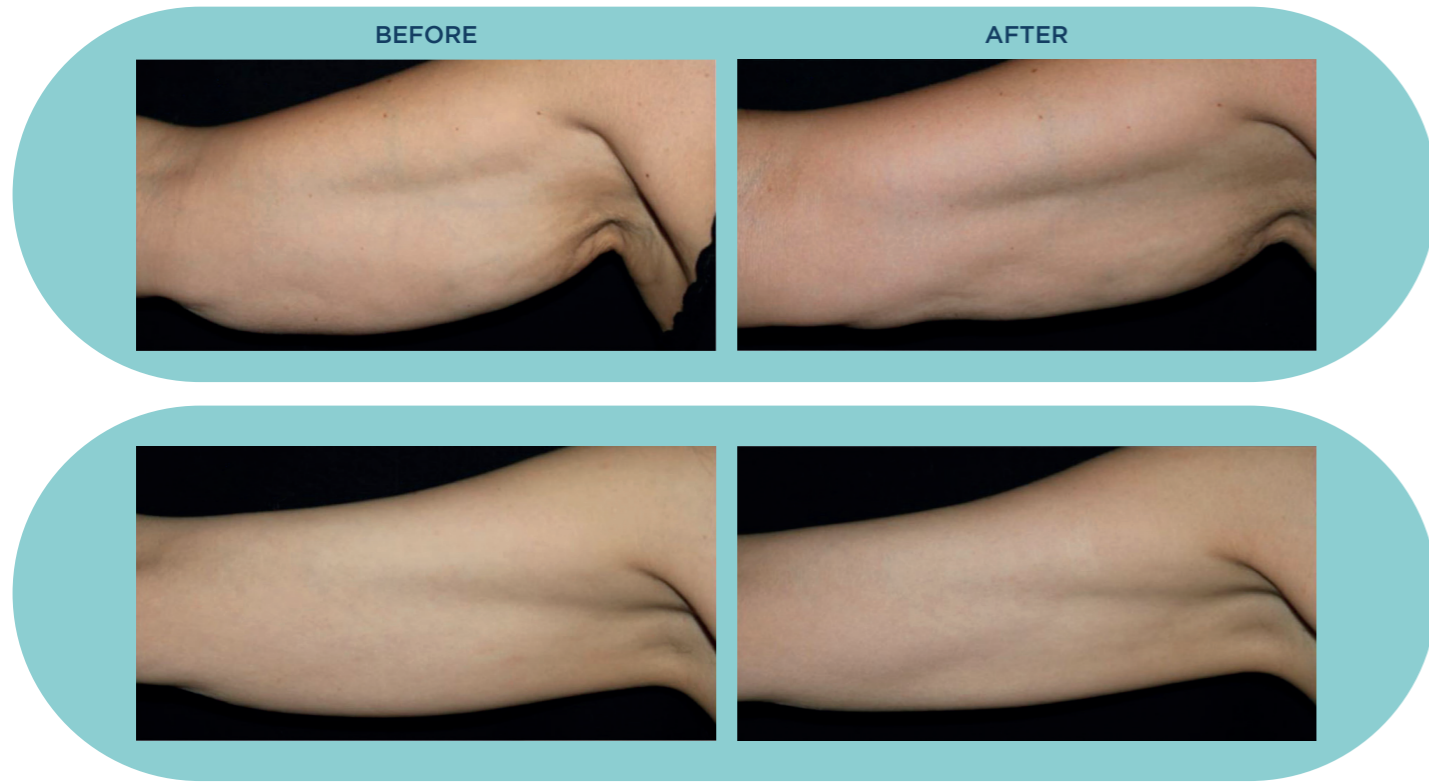

Material intended for medical practitioner's use only



**CE** 0373  
Class III Medical Device

Sunekos Body is composed of 10ml of a special low density HA + AA specifically balanced for the body.

[sunekos.com](http://sunekos.com)



Via Ciro Menotti, 1/A 20129 - Milano - Italy  
Tel: +39 02 744020  
[www.professionaldietetics.com](http://www.professionaldietetics.com)  
Email: [info@pd-international.com](mailto:info@pd-international.com)